

How to use a condom

If used properly, latex condoms are effective against pregnancy, AIDS, (HIV Infection), and other sexually transmitted diseases.

Caution: This product contains natural rubber latex which may cause allergic reactions.

MANDINGO[®]

CONDOMS

EFFECTIVENESS

MANDINGO[®] Condoms, if used properly, will help to reduce the risk of HIV infection (AIDS) and many other sexually transmitted diseases (STD'S), including chlamydia, genital herpes, genital warts, gonorrhea, hepatitis B, and syphilis. No prophylactic or contraceptive can guarantee 100% effectiveness. Also, MANDINGO[®] Condoms will help reduce the risk of pregnancy, while eliminating the serious side effects sometimes associated with other methods of pregnancy risk reduction.

The use of Male Latex Condoms does not totally eliminate the risk of contracting sexually transmitted diseases. Use of Male Latex Condoms for activity other than vaginal intercourse can increase the potential for breakage.

Caution: This product contains natural rubber latex which may cause allergic reactions.

Condom effectiveness can also depend directly on the lubrication involved during the sexual activity. The intensity of thrusting during sex, combined with the amount and slipperiness of the lubrication, can affect how well the condom stays on during sexual activity.

PRECAUTIONS

It is important to follow the directions shown here. Failure to follow directions for proper use may result in loss of protection when using MANDINGO[®] Male Latex Condoms. -During intimate contact, lesions and various body fluids can transmit STD's. This indicates that the condom should be applied before any such contact.

STORAGE

Male Latex Condoms should be stored at room temperature. Latex Condoms should be protected from extended periods of exposure to: extreme high or low temperature, moisture, direct sunlight, or fluorescent light.

DIRECTIONS FOR PROPER CONDOM USE:

- For maximum protection, the following are suggested precautions and directions for use:

Further Precautions:


- Use a new condom every time you have sexual intercourse or other acts between partners that involve contact with the penis.

- Do Not Reuse Condoms.
- Store in a cool, dry place.
- If the rubber material is sticky or brittle or obviously damaged, do not use it.
- If irritation or burning is experienced by either partner, discontinue use. Try another type of condom.
- If a lubricant is wanted, use water-based lubricants. DO NOT USE OIL-BASED LUBRICANTS, such as those made with petroleum jelly (e.g., Vaseline), mineral oil, vegetable oil, or cold cream, as these may damage the condom.
- If you use condoms where you or your partner are using vaginal products for medical treatment purposes, the condoms may be weakened and their effectiveness may be reduced.

FURTHER DIRECTIONS FOR USE:

-Tear open the package carefully. Do not use fingernails, teeth, or anything that can damage the condoms.

Step # 1

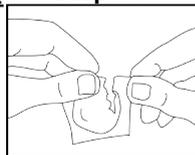

- Put the condom on after the penis is fully erect and before intimate contact.


Lesions, pre-ejaculate secretions, semen, vaginal secretions, saliva, urine, and feces can all transmit disease organisms. -If you have a foreskin, pull it back before rolling on the condom.

Step # 2




- Place the condom on the head of the penis. After unrolling over the head, squeeze tip of condom gently to remove any trapped air.

Step # 3

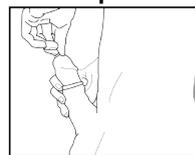


Step # 4

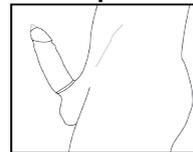

Then, unroll condom all the way to the base.
- Leave an empty space at the end of the condom to collect semen. The closed end of the condom has a nipple shaped end to allow for the collection of semen. Again, remove any air remaining in the tip of the condom by gently pressing the air out toward the base of the penis.

- After ejaculation and while the penis is still erect, hold on to the rim of the condom (the open end) so that the condom does not slip off as the penis is carefully withdrawn.
- Throw the used condom away. Wash up.

Step # 5

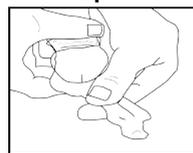

MANDINGO[®]

CONDOMS

GLOBAL FORCE ENTERPRISES LLC.
2331 W. Lake Miramar Circle
Miramar, FL. 33025 USA
www.mandingocondoms.com

MANDINGO[®] Condoms is a registered trademark of Global Force Enterprises LLC.

© 2012 All Rights Reserved